ENGLISH

IMPORTANT: These Instructions for Use are applicable to the contact lenses described below. Read carefully and retain for future reference. This document is intended for eye care professionals but must be made available to patients upon request. The eye care professional is responsible for providing appropriate instructions to the patient based on the prescribed lenses.

DEVICE IDENTIFICATION

Device type: Re-usable Soft (hydrophilic) Contact Lens
Material Name and Water content:

- Filcon 1b: Water content: 57%
- Filcon -TSP: Water content: 58%
- Methafilcon A: Water content: 55%
- Filcon V: Water content: 45%

Design: Sphere; Toric; Afocal; EDOF

Medical Device Classification: Class IIa (Regulation (EU) 2017/745)

Each lens is supplied sterile in a sealed blister containing buffered saline solution. The packaging indicates diameter, base curve, power (including cylinder and axis where applicable), LOT number and expiration date.

INTENDED USE

The contact lenses are intended for daily wear only (removal before sleep) for the optical correction of ametropia and, where specified, presbyopia, in individuals with healthy eyes. Lenses are designed for repeated use over a prescribed replacement schedule defined by the eye care professional (e.g., 7 days or 30 days).

INDICATION

The contact lenses are intended for the correction of refractive errors (myopia, hyperopia, astigmatism and/or presbyopia) in non-diseased eyes, to improve visual acuity.

REPLACEMENT SCHEDULE

Lenses must be replaced according to the manufacturer suggested schedule or as prescribed by the eye care professional, regardless of their apparent condition.

INTENDED USERS

End users trained in the proper use of contact lenses
Eye care professionals (opticians, optometrists, ophthalmologists)
Caregivers assisting contact lens wearers

TARGET PATIENT POPULATION

Individuals requiring vision correction with healthy eyes. Suitability must be determined by an eye care professional.

CLINICAL BENEFIT AND PERFORMANCE

Contact lenses improve visual acuity by correcting refractive errors while worn as intended.

Toric lenses correct astigmatism. Multifocal or EDOF designs support correction of presbyopia where applicable.

MODE OF ACTION

Contact lenses are designed to fit over the cornea. When positioned correctly, they refract and focus incoming light onto the retina, thereby improving visual acuity.

CONTRAINDICATIONS

Do not use contact lenses in the presence of any of the following conditions:

- Eye infections or inflammation
 - Redness, pain, or abnormal discharge
 - Any disease or abnormality affecting the cornea, conjunctiva, or eyelids
 - Severe dry eye
 - Known hypersensitivity or allergic reactions affecting the eye
 - Any systemic disease that may affect the eye or be exacerbated by contact lens wear
 - Active corneal infection (bacterial, fungal, viral or protozoal)
- Do not use damaged, contaminated, or compromised lenses.

PRECAUTIONS

Contact lenses are medical devices and must be fitted and prescribed by an eye care professional. Follow all instructions regarding wear schedule, replacement, cleaning, disinfection, follow-up visits and emergency procedures.

- Avoid wearing lenses when unwell, especially during cold or flu symptoms.
- Avoid contamination with cosmetics, creams, soaps, or aerosol products. Insert lenses before applying make-up and remove them before removing make-up.
- Do not expose lenses to water (tap water, bottled water, swimming pools, hot tubs, or the sea). Exposure increases the risk of infection. If exposed, remove, clean and disinfect before reuse.
- Avoid environments with harmful vapors or chemicals. Remove lenses immediately if exposed.
- Always remove lenses before sleep
- Always use lens care solution as advised by the eye care professional.
- Do not change lens type, parameters, or wearing schedule without professional advice.
- Avoid handling lenses with long fingernails.
- Always keep a pair of spectacles with a valid prescription available.
- Keep lenses out of reach of children who are not users.

WARNINGS

- Do not wear lenses beyond the prescribed replacement period.
- Do not sleep while wearing contact lenses.
- Do not use lenses beyond the expiration date.
- Do not use lenses if packaging is damaged or compromised.
- Do not use damaged, scratched, or contaminated lenses.
- Do not use saliva or non-sterile liquids to wet lenses.
- Do not wear lenses while using eye medications unless directed by a healthcare professional.
- Remove lenses immediately if exposed to harmful vapours or hazardous substances.
- Sharing contact lenses is prohibited and may cause serious eye infections.

If you experience discomfort, redness, pain, discharge, or vision changes, remove lenses immediately and consult an eye care professional.

Smoking increases the risk of serious ocular conditions such as ulcerative keratitis.

ADVERSE REACTIONS

The following symptoms may occur:

- Reduced comfort or foreign body sensation
- Burning, stinging, itching, or dryness
- Excessive tearing or abnormal discharge
- Redness
- Sensitivity to light (photophobia)
- Blurred or reduced vision
- Swelling of the eye or eyelids

If any of these symptoms occur, remove the lenses immediately. If symptoms persist, consult an eye care professional promptly. These symptoms may be associated with serious conditions such as corneal infection, keratitis, or corneal ulceration. Seek immediate medical attention if symptoms worsen.

RESIDUAL RISKS

Improper use of contact lenses may result in:

- Eye irritation
- Infection
- Corneal abrasions

In particular, inadequate hygiene, or failure to follow care instructions may result in:

- Microbial infection (including keratitis)
- Corneal ulceration
- Inflammatory reactions

Compliance with these instructions significantly reduces associated risks.

INSTRUCTIONS FOR USE

Handling

Wash and dry hands thoroughly before handling lenses. Ensure lenses are not inside out. Do not use damaged lenses.

Lens Insertion

Do not apply make-up until you have inserted your lenses. To avoid mixing up your lenses, insert them one by one. Make sure the lens is not inside out before insertion (see figures 1 & 2 for ref.). Place the lens on your index finger to verify shape. Using your other hand's index finger and second finger hold up the upper lid and hold down the lower lid. Place the lens on the eye (see figure 3). Repeat this step for the other eye.

Lens Removal

Look up and hold down the lower lid. Using your index fingertip and thumb tip to gently pinch the lens off the eye (see figure 4). Repeat this step for the other eye. For any doubt, please contact your eye care professional.

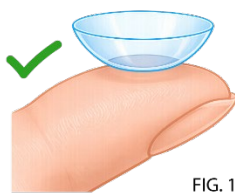
FIG. 1

FIG. 3

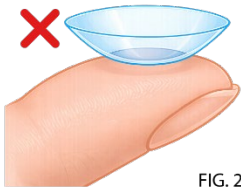
FIG. 2

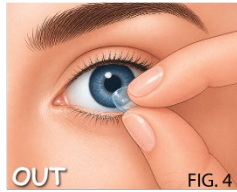
FIG. 4

LENS CARE AND MAINTENANCE

Lenses must be cleaned, rinsed, and disinfected after each use using a lens care solution as advised by the eye care professional.

Cleaning and Disinfection Procedure

After removal, place the lens in the palm - Apply solution and gently rub (if required by solution type) - Rinse thoroughly with solution - Place in a clean lens case filled with fresh solution - Soak for the minimum time specified by the solution manufacturer.

Storage Between Uses

Store lenses fully immersed in fresh disinfecting solution. Do not reuse old solution.

Lens Case Care

Empty and rinse lens case daily with solution - Allow to air dry - Replace lens case regularly (at least monthly)

Additional guidance

Do not "top up" old solution - Do not reuse solution - Do not use expired solutions - Do not mix different care systems unless advised.

STORAGE CONDITIONS

Store at room temperature (approximately +5°C to +25°C). Keep away from heat sources. Do not freeze.

DISPOSAL

Dispose of lenses after the prescribed replacement period. Dispose of used lenses and packaging according to local regulations. Do not dispose of in the environment.

EMERGENCIES

If chemicals enter the eye: **Rinse immediately with clean water**
Contact an eye care professional immediately or seek emergency medical care

REPORTING OF ADVERSE EVENTS

Any serious incident must be reported to:

Safilens S.r.l. (Manufacturer)
Via Maria Grazia Deledda, 5
34079 - STARANZANO (GO)
ITALY
Phone nr.: +39 0481 480421
email: report@safilens.com

Symbols and Abbreviations:

 Safilens S.r.l. Via Maria Grazia Deledda, 5 IT34079 Staranzano ITALY	Manufacturer name and address
	CE Mark
	Warning
	Sterilization using dry heat or steam (autoclave)
	Medical Device
	Read instruction before use
	Do not use if packaging is damaged
	Single sterile barrier system
SPH ; BC ; DIA ; CYL ; AXIS	Spherical dioptric power, lens base curve, lens diameter, lens cylinder and lens axis (for toric products)
	Product lot number
	Product Expiry date – use by date
	Product manufacturing date
   	Recycling symbols

21 Astuccio carta Paper box
22 Foglietto carta Paper leaflet IFU
02 Flacone e Portallenti Bottle and Lens carrier
04 Sottotappo Bottle Pin

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